

SureSmile® Aligner

Using the SureSmile® VPro™ App

SureSmile®
Clear Aligners



- STEP 1** Download the SureSmile VPro App.
Ensure the SureSmile VPro is charged and within range of phone.



- STEP 2** The SureSmile VPro will need to be paired to your phone via Bluetooth.
Open the app and continue to the “Pair Device” screen. Input credentials including the email provided by treating clinician.

- STEP 3** Click “SELECT DEVICE”, choosing the device ID starting with the word VPRO, click “SELECT VPRO DEVICE”.

- STEP 4** Navigate back to the Pair Device screen. Click “GET STARTED”.

Your SureSmile® VPro™ is now paired.



Device: VPRO_B47127EDC149

Austin

Powers


Dr.Jones@DrJones.com

PAIR NEW DEVICE


SET ALIGNER CHNGE INTERVAL

GET STARTED


WEEKLY VIEW

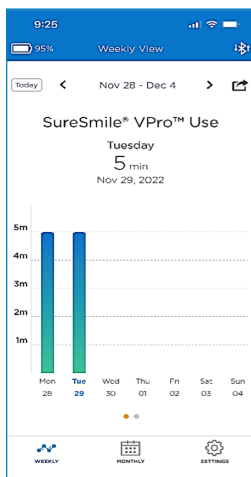

MONTHLY VIEW

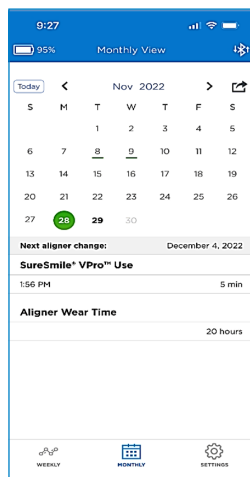

ALIGNER USAGE

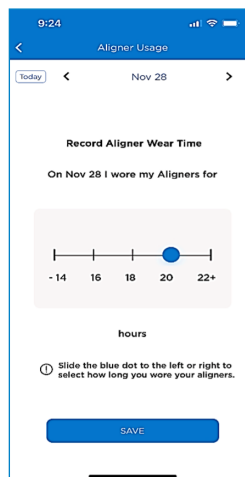

- The SureSmile VPro app automatically collects the data after each daily session, even recording the data when not in range/ Bluetooth is off and sending the data when there is a connection at the next time of use.
- Aligner Wear should be manually entered. The Aligner Change Reminder will vary depending on what change interval the clinician prescribes.

- STEP 5** Progress photos can be uploaded by navigating to “Settings” and selecting “Progress Photos”. Click “SHARE” to send photos to the provided email or simply save them to the camera roll.

